



**Floseal**  
*Haemostatic Matrix*



## Floseal application technique <sup>1</sup>


1

Apply Floseal directly to the site of bleeding, allowing it to conform to the shape of the wound




2

Apply Floseal at a faster rate than the bleeding to prevent product from being washed away due to brisk bleeding

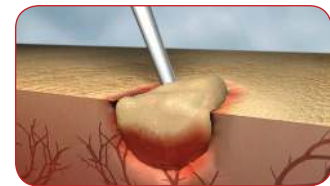

3

Keep Floseal at the site of bleeding using a damp swab and gentle approximation

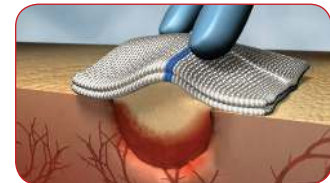

4

Wait 2 minutes before inspection (brisk bleeding/coagulopathic patients may require longer).  
In cases of persistent bleeding, reapply Floseal through existing material to ensure newly applied Floseal reaches the bleeding site itself



5

Once haemostasis is achieved, gently irrigate away excess granules not incorporated into the clot



6

Floseal is reabsorbed by the body within 6–8 weeks, consistent with normal wound healing

