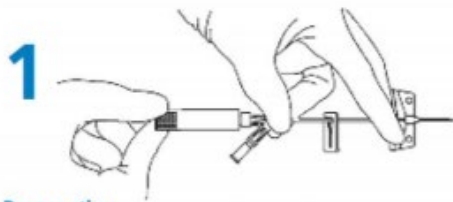


INSERTION OF BD SAF-T-INTIMA™ DEVICE


BD Saf-T-Intima™ for Subcutaneous infusion therapy

**1**
Preparation

- Hold as shown (Fig. 1) and rotate the white safety shield to loosen the needle. (Fig. 1).

**2A**
2B
Insertion

- Grasp the textured sides of wings and bring them together, pinching firmly. (Fig. 2A).
- Using thumb and index finger gently pinch the skin around selected site to identify the subcutaneous tissue. (Fig. 2B).
- Insert the full length of the catheter and needle through the skin at a 30°-45° angle. (Fig. 2B).

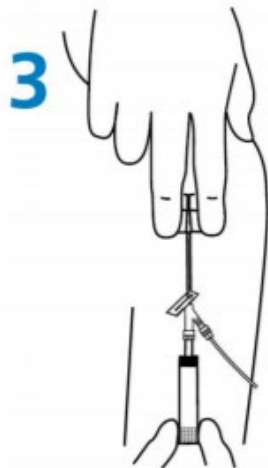**3****Needle Removal**

- Lay the wings flat on the skin surface and pull the white safety shield in a straight, continuous motion until the safety shield separates from the safety system. (Fig. 3).
- Discard the needle immediately in a puncture resistant, leak-proof sharps container.

**4****Stabilisation**

- Secure the catheter and apply a sterile dressing per facility protocol.